waterpik[®] SinuSense[™]

QUICK START GUIDE

SOOTHING SALINE EASY-POUR PACK


ENGLISH

1 Tear and pour the Soothing Saline Easy-Pour Pack into the Waterpik[®] SinuSense[™] device (Neti Pot, Neti Bottle, Squeeze Bottle or Water Pulsator).

2 Fill the bottle or reservoir to the top fill line with 8oz (240 ml) of warm distilled water or previously boiled water. Tighten the cap or fasten reservoir to the device and gently shake to mix the solution.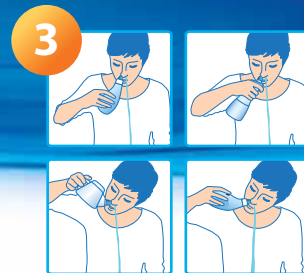

3 While at the sink, lean forward and fit the tip of the Waterpik[®] SinuSense[™] device nozzle inside the nostril forming a seal to avoid any leakage. Open your mouth slightly. Control the water flow by engaging the FlowControl[®] vent, squeezing the bottle or pulling gently on the FlowControl[®] trigger. During this sinus washing procedure, breathe continuously through your open mouth to allow a necessary air passageway so that the water will not drain from behind your nose into your mouth. Do not inhale or snort the solution into the nose. Allow approximately 4 ounces or half of the solution to clean and moisturize. Exhale through both nostrils and gently blow your nose to remove any excess water from your nose. Repeat on opposite side of nasal passage. Thoroughly clean your device after each use.

Warnings

- Keep out of reach of children.
- Do not use if nasal passage is completely blocked or if you have an ear infection or are experiencing pressure in the ear.
- For use by 1 person only.
- Do not swallow.
- Closely instruct and supervise children under 12 years of age and individuals with special needs in the proper use of this product to minimize swallowing.
- If you experience any discomfort, discontinue using this product and consult a health care practitioner.
- Consult a health care practitioner if symptoms persist or worsen.
- Refer to your device instruction manual for complete details.

Go to www.waterpiksinusense.com to learn more about how to use Waterpik[®] SinuSense[™] products or refer to your device instruction manual for more information about how to use.

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© 2012 Water Pik, Inc.
FN 20014354-F AB